NeoRhIn Baby

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⚠️ Výstraha.

NeoRhIn Baby:

Please read these instructions for use carefully as they contain important information. NeoRhIn Baby is a medical device. When using this device, please follow these instructions for use. Keep this leaflet in case you need to read it again.

Intended purpose: NeoRhIn Baby is intended for children from the age of 3 months. The product is applied into the child's nostrils in the form of a spray by pressing a mechanical sprayer to evenly cover the nasal mucosa. NeoRhIn Baby is used:

- to hydrate dry nasal mucosa and clear the nose, thereby reducing the risk of infection,
- for the treatment of colds and nasal congestion without the risk of addiction and damage to nasal mucosa,
- to reduce the risk of complications associated with colds (sinusitis, inflammation of the middle ear (otitis media)).

Ingredients: Hypertonic sea salt solution (1,1 g / 100 ml) with a higher content of trace elements (Mg, Mn), dexpanthenol (1 g/100 ml), hydroxypropyl methylcellulose, disodium edetate, aqua purificata. The product does not contain blood derivatives.

Characteristics: The product's main active component is a hypertonic solution of salt from Dead Sea, which hydrates the nasal mucosa, mechanically cleans their surface and helps reduce swelling on the principle of osmosis, which clears the nose. Panthenol has a soothing effect on irritated nasal mucosa.

Indications: NeoRhIn Baby is intended for children from the age of 3 months to clear the nose, hydrate nasal mucosa and provide relief from colds and nasal congestion.

Contraindications: The nasal spray must not be used in case of known hypersensitivity to any of the ingredients or in case of respiratory illnesses accompanied by pronounced hypersensitivity. Do not use after nasal surgery or trauma in the area of the nose.

Interactions: When used in conjunction with products designed to reduce the swelling of nasal mucosa, use this product first. For more serious complications requiring the short-term application of topical products, it is recommended you continue treatment with NeoRhIn Baby nasal spray for several days after the end of this application until complications subside.

Adverse effects: In rare cases, a reaction may occur due to hypersensitivity, which presents as a redness of the nostrils.

Dosage and Method of Application: The product is intended for children from the age of 3 months. The nasal spray can be used long-term (more than 30 days) to clear nasal passages and to support the treatment of illnesses due to colds. If complications persist, you should consult your doctor.

Shake the bottle well and remove the cap from the sprayer before use. The dosage for each indication is performed in the same way and repeated several times a day, as needed (until symptoms subside). The number of daily doses is not limited. Compress the mechanical sprayer one to three times for each application.

The product should be applied by an adult for children up to the age of 6.

Children aged from 3-6 months are placed on their back with their head facing forward and the product is applied with one spray into each nostril. When using the nasal spray in a horizontal position, the applicator should be turned into a vertical position between each dose, directed away from the child and pressed to draw the solution into the sprayer.

After use, rinse the sprayer with lukewarm water and replace the cap. Before using the sprayer for the first time, press the sprayer several times to draw the solution into the applicator.

Warning: In case of eye contact, rinse with water. One pack of the product should not be used by more than one user to avoid the spread of colds.

Storage: Keep away from children. Store in the original package.

Caution: The product must not be used after the expiration date. Once opened, do not use the nasal spray for longer than 30 days.

Packaging: NeoRhIn Baby is sold in a brown glass bottle with a mechanical spray applicator. The bottle contains 30 ml of solution.

Disposal: Return any unused product to the pharmacy. Dispose of the clean packaging in the appropriate recycling container.

Neofyt spol. s r.o., Pražská 442, 281 67 Stříbrná Skalice, Czech Republic, e-mail: phyteneo@phyteneo.cz, www.phyteneo.cz

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